USO PREVISTO:

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INTENDED USE:

PORTUX 3D SG is a material specifically designed for the 3D printing of precise and stable dental surgical guides. This material must be handled exclusively by dentists or dental technicians who are properly trained in the use of photopolymerizable resins for 3D printing.

COMPOSITION:

The resin is a mixture of:

- Acrylic monomers.
- Photoinitiators (Diphenyl(2,4,6-trimethylbenzoyl)phosphine oxide).
- Additives.
- Dyes.

DESCRIPTION:

PORTUX 3D SG is a low-viscosity, photopolymerizable resin specifically designed for the 3D printing of sterilizable surgical guides. It offers excellent dimensional accuracy and high definition, ensuring faithful reproduction of the digital design.

Post-cured and sterilized guides exhibit a translucent light-blue color, free from yellowing or chromatic degradation, providing optimal visibility of adjacent tissues and structures during clinical procedures.

PORTUX 3D SG is compatible with LCD/DLP stereolithography 3D printers operating at wavelengths of 385 and 405 nm.

INSTRUCTIONS FOR USE:

- Mix the product for at least 20 minutes using a mechanical shaker or roller before first opening. This ensures proper printer performance and consistent color reproduction.
- To minimize bubble formation, agitate the product at least 1 hour prior to use.
- Print with the resin following your printer's handling and operating instructions.

Post-processing of printed guides:

- Clean the printed structures in isopropyl alcohol or ethanol (>96%), preferably using an ultrasonic bath to facilitate cleaning. Immerse the prints in a container with alcohol for 5 minutes. Then, submerge them in another container with fresh alcohol for the same duration. It is recommended to gently apply compressed air between cleaning steps to remove excess resin from cavities or critical areas of the print.
- Remove the structures from the alcohol bath and dry them either by gently applying compressed air or by placing them in an oven at 40 °C for 30 minutes. **IMPORTANT:** Do not cure wet or moist pieces, as this will compromise the dimensional accuracy and final definition of the printed structure. After cleaning and drying, place the printed parts in the NextDent LC-3DPrintbox curing unit and cure them as follows:

- First exposure: 15 minutes.
- Return the piece to the printer and allow the chamber to cool for 5 minutes.
- Second exposure: 15 minutes.
- If using a photopolymerizable glaze, the curing times specified by the manufacturer must be applied during either the first or second exposure, without adding to the previously defined exposure times.

After post-curing, it is recommended to sterilize the surgical guide in an autoclave using steam at 121°C for 15 minutes.

WORKING IN CONJUNCTION WITH OTHER DEVICES:

Surgical guides manufactured with PORTUX 3D SG resin must be previously designed in surgical planning software, taking into account the type and brand of the implant to be used during the procedure. The choice of implant will determine the brand and type of ring to be incorporated into the guide, as well as the corresponding guided surgery kit (consisting of drills and spoons) that must be used during the intervention.

RESIDUAL RISKS:

- Contact with uncured material or the release of unpolymerized components may cause allergic reactions. This risk can be minimized by performing post-curing for the time and under the conditions specified. Biocompatibility testing supports the safety of the device during its intended use.
- There is a risk of infection if the device is not sterilized prior to its insertion into the patient's oral cavity. This risk is prevented by following the indicated sterilization parameters.
- 3D-printed surgical guides may present dimensional discrepancies with the ring and/or the guided surgery kit. To avoid this, it is essential to perform accurate design in the planning software and to

correctly parameterize the resin in the printing equipment.

-Extreme conditions during the sterilization process may cause fractures or deformations in the printed guide, rendering it unusable. This risk can be mitigated by strictly complying with the established sterilization parameters.

-Improper storage of the resin may alter its polymerization properties, making it unsuitable for use. To prevent this, store the product correctly, protect it from light, and return it to its original container immediately after use.

CONTRAINDICATIONS:

The use of surgical guides manufactured with PORTUX 3D SG resin is contraindicated in patients with a confirmed history of allergic reactions to methacrylates or other acrylic resin-derived compounds.

WARNINGS:

This product must not be used for any application other than that specified in its intended use. Surgical guides printed with PORTUX 3D SG are biocompatible for contact durations of less than 24 hours, provided that the instructions for use set forth in this document are followed. They must not be used for periods longer than those specified.

PRECAUTIONS:

Use in children, pregnant or lactating women is not recommended.

STORAGE:

The product should always be kept within its original packaging, at room temperature (< 30°C) avoiding the following conditions:

- Direct exposure to sunlight or other types of radiation.
- High heat or humidity sources.
- Dust or any other type of contaminant.

FINAL DISPOSAL:

Do not dispose this product in water sources or sewage systems. This product should not be discarded in soil or subsoil. Disposal should be carried out through an authorized waste management company, in accordance with national and local regulations.

SHELF LIFE:

3 years.

 Fabricado por/Manufactured by
New Stetic S.A. Cra 53 N° 50-09 Guarne - Antioquia-Colombia.
Tel: (+ 57-604) 403 8760 - 550 0000
infocolombia@newstetic.com / www.newstetic.com

	Consult instructions for use
	Maximum storage temperature
	Caution: Consult the accompanying documents. See the instructions for additional warnings or suggestions for proper use of the device.
	Keep away from sunlight
	Wear gloves
	Caution: The product may cause respiratory irritation, drowsiness, allergic skin reactions, and eye irritation
	Caution: Hazardous to the environment
	Keep in a dry place
	Eye protection