### PMMA CAD CAM Disc

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Elaborado: 2019-10-25 / Actualizado: 2023-10-20

Finishing and polishing are carried out according to the usual procedures.

#### WORK WITH OTHER DEVICES:

It is used over other devices such as dental implants, bolts and preparations. The device is cemented over such structures. The product is compatible with dental acrylic resins.

#### RESIDUAL RISKS:

See contraindications, warnings and cautions associated with the product use.

#### CONTRAINDICATIONS:

In case of suspected intolerance, use the product only after having excluded the presence of allergies with a consultation with the allergist.

#### WARNINGS:

Avoid overheating during milling and finishing. The provisional can be fixed with all the cements for temporary fixation. Provisional cements with Eugenol can decrease the effectiveness of resinous cements.

For professional dental use only

#### CAUTIONS

The hygiene and the correct use of the prosthesis is the responsibility of the patient. The patient is responsible for the correct use of the product. The manufacturer is not responsible for any unsatisfactory results since it cannot influence the manufacturing process in any way.

Avoid excessive wear on the dental structure obtained when making the final adjustment on the patient. This could lead to growth and accumulation of plaque and microorganisms.

The dental technician must inform the dentist about all the operations to be carried out before placing the piece in the patient's mouth, making sure that the professional has received all the information contained in these instructions for use. The request for compensation for eventual damages may refer exclusively to the commercial value of our product.

#### STORAGE:

Store in a fresh and dry place. Keep away from sources of ignition.

#### FINAL DISPOSAL:

The product disposal must be done according to the local guidelines.

#### SHELF LIFE:

Ten (10) years.

In case of serious accidents, contact: dosorio@newstetic.com (Technical Director of Medical Devices) and the email infocolombia@newstetic.com

#### New Stetic S.A.

Carrera 53 N° 50 - 09, Km 22, autopista Medellín - Bogotá.  
Guarne - Antioquia - Colombia.  
(+57-604) 5500000  
[www.newstetic.com](http://www.newstetic.com) - [infocolombia@newstetic.com](mailto:infocolombia@newstetic.com)

Consult instructions for use	Keep in a dry place
Storage Temperature	Caution: Consult accompanying documents. Consult additional warnings or cautions included in the instructive for a correct use of the device.

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