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PMMA CAD CAM Disc
PMMA discs for manufacturing of dental structures with CAD CAM technology.

The PMMA discs are directly used by the dentist or dental technician for the manufacturing of dental structures with CAD/CAM technology. The variants of the product include the following applications:

- Provisional anterior and posterior crowns & bridges.
- Implant supported prosthetics.
- Partial, complete and hybrid denture prosthetics (base and teeth).
- Removable appliances (splint).
- Copy of dental metallic structures by casting method.

The product has excellent properties for long term provisional use.

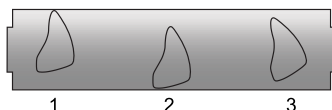
COMPOSITION: Polymethyl methacrylate, biocompatible pigments, EDMA and fluorescence

The discs for CAD/CAM application produced by New Stetic are made of poly methyl methacrylate (PMMA), a known material used in the dental field due their physical-chemical, mechanical and biocompatibility properties. These discs are intended for dental CAD/CAM use and have a standard format according to the commercial CAD/CAM machines specifications. Also, the CAD/CAM discs have a wide variety of colors and shapes.

INSTRUCTIONS FOR USE

Place the disc in the machine following the instructions of the equipment manufacturer. Consider specific parameters for milling acrylic polymers. The intensity of the incisional region of the restoration is determined by the vertical position of the restoration in the disc. If your CAD/CAM software supports it, adjust the vertical position of the restoration in the CAM software project set up to achieve the desired results.

After machining, remove the disc from the equipment and remove the milled element from the block. For finishing, use milling cutters, abrasive strips and rubber wheels. If gloss is necessary, use mechanical polishing. Carefully check the finished restoration for obvious defects. If a defect is found, do not use restoration.

It is important to highlight that to ensure the intensity of the incision in the design of the case, the positioning of the axis perpendicular to the layer of the block is essential. For multilayer disc only.

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Elaborado: 2019-10-25 / Actualizado: 2023-10-20

Finishing and polishing are carried out according to the usual procedures.

WORK WITH OTHER DEVICES:

It is used over other devices such as dental implants, bolts and preparations. The device is cemented over such structures. The product is compatible with dental acrylic resins.

RESIDUAL RISKS: See contraindications, warnings and cautions associated with the product use.

CONTRAINDICATIONS: In case of suspected intolerance, use the product only after having excluded the presence of allergies with a consultation with the allergist.

WARNINGS: Avoid overheating during milling and finishing. The provisional can be fixed with all the cements for temporary fixation. Provisional cements with Eugenol can decrease the effectiveness of resinous cements. For professional dental use only

CAUTIONS

The hygiene and the correct use of the prosthesis is the responsibility of the patient. The patient is responsible for the correct use of the product. The manufacturer is not responsible for any unsatisfactory results since it cannot influence the manufacturing process in any way.

Avoid excessive wear on the dental structure obtained when making the final adjustment on the patient. This could lead to growth and accumulation of plaque and microorganisms.

The dental technician must inform the dentist about all the operations to be carried out before placing the piece in the patient's mouth, making sure that the professional has received all the information contained in these instructions for use. The request for compensation for eventual damages may refer exclusively to the commercial value of our product.

STORAGE: Store in a fresh and dry place. Keep away from sources of ignition.

FINAL DISPOSAL: The product disposal must be done according to the local guidelines.

SHELF LIFE: Ten (10) years.

In case of serious accidents, contact: dosorio@newstetic.com (Technical Director of Medical Devices) and the email infocolombia@newstetic.com.

New Stetic S.A.
Carrera 53 N° 50 - 09. Km 22, autopista Medellín - Bogotá.
Guarne - Antioquia - Colombia.
(+ 57-604) 5500000
www.newstetic.com - infocolombia@newstetic.com

Consult instructions for use	Keep in a dry place
Storage Temperature	Caution: Consult accompanying documents. Consult additional warnings or cautions included in the instructive for a correct use of the device.

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