USO PREVISTO

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INTENDED USE

Production of individual trays for taking definitive impressions of the ridge in totally or partially edentulous patients who require a dental restoration.

COMPOSITION

Monomer: Methyl methacrylate (MMA).
Polymer: Polymethyl methacrylate (PMMA).

GENERAL DESCRIPTION

The product is a self-curing acrylic resin based on PMMA, used for the elaboration of individual trays that allow the taking of impressions of the patient's ridge when they require a dental restoration as a result of the partial or total loss of the natural teeth. The tray is introduced into the patient's mouth containing the impression material that will make the copy of the oral tissue and that will later allow to obtain the model on which the definitive restoration will be worked.

INSTRUCTIONS FOR USE

- Mark the bottom of the sulcus without invading the frenulum area.
- Mold a wax base plate on the plaster model and trim the excess without reaching the bottom of the sulcus.
- Apply petroleum jelly to the tiles and place the spacer stops.
- Pour the polymer over the monomer in the following proportions:

By volume: 3 parts polymer and 1 part monomer.
By weight: 2 parts polymer and 1 part monomer.

- Mix crosswise for approximately 30 seconds, to avoid the generation of air bubbles, and to ensure that the polymer particles are completely incorporated with the monomer.
- Cover the container to avoid the inclusion of air until the mixture is in the plastic stage (the mixture does not adhere to the spatula).
- Place the mixture in plastic phase on the glass tiles and press until the spacer stops that delimit the thickness make contact.
- Mold the acrylic resin sheet over the preliminary model and remove excess.
- Fix the acrylic resin sheet until polymerization begins.
- Remove the base wax from the interior of the individual tray.
- Polish excess acrylic resin to the limit of the initial marking on the model.
- Make the handle of the individual tray and placed it on the rim at an angle of approximately 45 degrees. It should not encroach on the bottom of the groove and should be sized to allow the professional to hold the tray firmly during its handling.

For a better finish, polish according to the conventional procedure used in the laboratory.

RESIDUAL RISK

Release of residual methyl methacrylate monomer that could cause allergy or irritation to the oral mucosa, a situation that is avoided by maintaining the polymer-monomer ratio indicated for the use of the product. However, the product is for transient use which minimizes the risk of any adverse reaction.

CONTRAINDICATIONS

The product should not be used in patients with hypersensitivity or allergy to the material.

WARNINGS

Monomer is a flammable liquid and should be used away from sources of sparks, flames, or high temperatures.

PRECAUTIONS

- The use of solvents on acrylic is not recommended because it may cause micro-fractures or cracking of the material.
- Do not use the product after the expiration date.
- Keep hands and working instruments dry to avoid incorporating bubbles into the acrylic.
- It is recommended to work in ventilated areas, preferably with a fume extraction system, protective goggles, latex gloves, and apron.
- Avoid contact of dust with eyes and avoid inhalation of dust, or inhalation of vapors.
- In case of contact of the powder or monomer with the eyes, the eyes should be rinsed with plenty of water. In case of contact of the monomer with the skin, the skin should be washed immediately with plenty of water and soft soap.
- In case of inhalation remove the affected person from exposure and provide fresh air. If required, give oxygen or artificial respiration, or consult a physician.
- In case of ingestion induce vomiting and seek medical attention immediately.
- For more information, please refer to the product safety data sheet at www.newstetic.com.

Failure to follow the recommendations in these instructions for use may lead to impairment of product performance.

RECOMMENDATION

In order to ensure the functionality of the product, use New Stetic self-curing monomer.

STORAGE

The product should be stored in a cool, dry, and clean place, away from heat sources and direct sunlight, at a maximum temperature of 30 °C.

FINAL DISPOSAL

Product residues and contaminated containers may be disposed of by incineration, always observing the local regulations in force. It should not be disposed of in water sources. After use of the device on the patient, it should be disposed of as special biohazardous waste.

SHELF LIFE

Polymer: 4 years.
Monomer: 3 years.

Liquid: INVIMA2017DM-0000632-R2
Powder: INVIMA2017DM-0000614-R2

Product only for dental use by qualified personnel.
Keep out of reach of children.

In case of serious accidents contact: dosorio@newstetic.com (Technical Director of Medical Devices) and infocolombia@newstetic.com.

SYMBOL GLOSSARY

 Instructions for use	 Fragile
 Keep dry, away from humidity sources	 Flammable
 Keep away from sunlight or direct heat sources.	 Irritant
 Maximum allowable storage temperature.	 The use of nitrile gloves is suggested for handling the product.

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New Stetic S.A.