ZAFIRA

Universal Bond

USO DESTINADO

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Universal Bond

INTENDED USE

It is a product for professional dental use, indicated for direct and indirect restorations with light-curing materials and for the treatment of cervical sensitivity. The main product indications are:

- Direct-placed light-curing composite and ceromer restorations.
- Direct-placed core build-ups with light-curing composites.
- Repair of fractured composite and ceromer restorations.
- Sealing of cavities prior to amalgam restorations.
- Immediate dentin sealing of prepared tooth surfaces before temporary/permanent cementation of indirect restorations.
- Adhesive cementation of indirect restorations with light-curing luting composites.
- Protective varnish for glass ionomer-based fillings.
- Desensitization of cervical areas.

COMPOSITION

Zafira Bond® contains methacrylates (UDMA, TEGDMA, BISGMA, GDMA and HEMA), 10-MDP, ethanol, water, dispersed silicon dioxide, initiators and stabilizers.

DESCRIPTION

Zafira Bond® is a light-curing single-component dental adhesive that combines etching, primer and bonding in one bottle. Zafira Bond® is an ethanol/water-based dental adhesive which bonds to dentin and enamel. The previous use of an etchant is optional in both, enamel and dentin; therefore, the adhesive can be used in three ways: self-etching, selective enamel etching or total etching (enamel and dentin). The product can be used to bond direct and indirect restorations of light-curing composite and ceromer to dentin and enamel.

INSTRUCTIONS

1. Cleaning: Clean the prepared cavity with water or a rubber cup and pumice or fluorine-free prophylaxis paste. Wash thoroughly with water spray and air dry (do not desiccate).

2. Isolation: A dry operating field must be ensured, can be using a rubber dam or cotton rolls and a saliva ejector.

3. Pulp protection: In very deep cavities, areas close to the pulp must be coated with a calcium hydroxide material or glass ionomer as a pulp protection base

For Self-Etch Technique, omit Steps 4-5 and proceed to Step 6.

4. Conditioning with phosphoric acid gel:

For Selective Enamel Etch Technique:

4.A. Etch enamel margins with a phosphoric acid gel etchant (e.g., Zafira Etchant Gel) for 15 to 30 seconds, then rinse thoroughly with water ensuring that all etchant has been removed. Proceed to Step 5.

For Total Etch Technique:

4.B. Etch enamel and dentin cavosurfaces with a phosphoric acid gel etchant (e.g., Zafira Etchant Gel 37%) for 15 seconds, then rinse thoroughly with water (twice the etching time) ensuring all etchant has been removed. Proceed to Step 5.

5. Drying: Dry lightly with water and oil free air; do not desiccate.

6. Application of Zafira Bond: Shake Zafira Bond bottle briefly prior to use.

Note: Upon first use, shake Zafira Bond bottle vigorously for 10 seconds.

6.1. Procedure for direct and indirect restorations with light-curing composites and ceromers, root surface desensitization, sealing of cavities before filling with amalgam, sealing of cavities and stump preparations before temporary attachment of indirect restorations.

- Dispense 1-2 drops of Zafira Bond into a clean container or, if preferred, directly onto a brush or disposable applicator. Protect the adhesive from light. Close the bottle immediately after dispensing so that the solvent does not evaporate.
- Using the brush or disposable applicator, apply a Zafira Bond's coat on the tooth surface, rubbing vigorously for 20 seconds and aerating with a gentle current of air until the solvent completely evaporates. The surface should have a smooth, shiny appearance. Do not polymerize.
- Apply a second layer of adhesive with a brush or disposable micro applicator on the surface of the tooth, rubbing vigorously for 10 seconds and airing. The surface should have a smooth, shiny appearance.
- After applying the second layer, light cure for 20 seconds using a lamp with a light intensity ≥ 800 mW/cm².
- For the remainder of the procedure, the instructions for use for the restorative and cementation materials must be observed.

6.2. Procedure for intraoral repair of fractured composite and ceromer restorations with light-curing materials

- Roughen the restorative surface to be repaired with a diamond finishing bur, clean with water spray, and dry with oil- and moisture-free compressed air.
- The subsequent working steps are the same as the application used for direct/indirect restorations with Zafira Bond including the acid conditioning (Step 4).

6.3. Procedure for use as protective varnish for glass ionomer fillings

- Place glass ionomer-based cement/restorative according to the manufacturer's instruction.
- Dispense 1-2 drops of Zafira Bond into a clean container or, if preferred, directly onto a brush or disposable applicator. Protect the adhesive from light. Close the bottle immediately after dispensing so that the solvent does not evaporate.
- Apply a layer of adhesive with a brush or disposable micro applicator on the surface of the tooth, rubbing vigorously for 20 seconds and aerate with a gentle current of air until the solvent completely evaporates. The surface should have a smooth, shiny appearance. Do not polymerize.
- Apply a second layer of adhesive with a brush or disposable micro applicator on the surface of the tooth, rubbing vigorously for 10 seconds and airing. The surface should have a smooth, shiny appearance.
- After applying the second layer, light cure for 20 seconds using a lamp with a light intensity $\geq 800 \text{ mW/cm}^2$.
- Remove the oxygen-inhibited layer with an alcohol-moistened cotton pellet or gauze.

CONTRAINDICATIONS

Zafira Bond® is contraindicated for:

- Direct pulp cappings.
- Use with patients who have a known allergic reaction to methacrylate resins or any of the components.
- If the stipulated working technique cannot be employed.

WARNINGS

- Zafira Bond® contains substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to methacrylate monomers or any other components in the product. If contact with skin occurs, wash immediately with soap and water.
- Contact with the eyes can cause serious eye damage. As prevention, wear protective glasses. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- If prolonged contact with the oral mucosa occurs, rinse thoroughly with water. If sensitization of mucosa persists, seek medical attention immediately.
- Zafira Bond® contains flammable components therefore it must be kept away from heat, sparks, flames or any other source of ignition.
- Dental materials contains Eugenol should not be used in conjunction with Zafira Bond® as it can affect the polymerization of the adhesive monomers.

CONSERVATION AND STORAGE

- Do not expose restorative material to elevated temperatures or intense light.
- Close the bottle immediately after use, to avoid solvent evaporation.
- Store the product in a clean, dry place, protected from light, in its original packaging, away from heat and at a maximum temperature of 30°C .
- Product shelf life is 3 years from manufacturing date (See expiration date on bottle and packing).
- Do not use the product after expiration date. - Do not expose restorative material to elevated temperatures or intense light.
- Close the bottle immediately after use, to avoid solvent evaporation.
- Store the product in a clean, dry place, protected from light, in its original packaging, away from heat and at a maximum temperature of 30°C .
- Product shelf life is 3 years from manufacturing date (See expiration date on bottle and packing).
- Do not use the product after expiration date.

FINAL DISPOSAL

Dispose of contents/container in accordance with local regulations. Incinerate uncured product at a licensed waste incineration facility. As a disposal alternative, use a permitted waste facility. Waste product that has been fully polymerized may be placed in a landfill designed for industrial waste. Consult with the respective regulatory authorities to determine possible treatment and disposal facilities.

EXPIRATION TIME

3 years.

HEALTH CERTIFICATE

INVIMA 2020DM-0022030.

**PRODUCT RECOMMENDED ONLY FOR DENTAL USE BY QUALIFIED PERSONNEL.
KEEP OUT OF THE REACH OF CHILDREN**

 See instructions for use	 Keep away from sunlight or direct heat sources.
 Manufacturer	 For USA Only (Federal Law USA) restricts this device to sale by or on the order of a licensed dentist.

 Manufactured by: New Stetic S.A.
Cra. 53 No. 50-09 Guarne, Antioquia - Colombia
infocolombia@newstetic.com - www.newstetic.com